

BLADDER TURBT TISSUE

SPECIMEN AND BLOCK SELECTION

1. Verify tumor pathology report indicates definitive diagnosis of urothelial carcinoma.
2. Select TURBT specimen with tumor used to establish bladder cancer diagnosis.
3. Select H&E slide and FFPE block most representative of the AJCC stage I to IIIA bladder cancer.

SAMPLING INSTRUCTIONS

Please select one of the following options:

OPTION 1: BLOCK

1. Submit the selected FFPE block.

OPTION 2: SLIDES

1. Submit one (1) H&E slide and nine (9) 5µm thick serially sectioned unstained slides or ten (10) 10µm thick serially sectioned unstained slides.
2. Label the serial sectioned slides (1-10) in the order in which they were cut.
3. Store unstained slides at 4 degrees C until shipping.

SHIPPING REQUIREMENTS

To avoid damaged or contaminated samples please use Veracyte's Decipher specimen kits along with the ice packs we provide. To obtain kits or shipping account information, please contact us at 1.888.792.1601 or specimenprocurement@veracyte.com.

OPTION 1: BLOCK

1. Place the selected FFPE block into the small zip-lock bag.
2. Place the small zip-lock bag containing the block into the foam insert.

OPTION 2: SLIDES

1. Place slides into the slide holders provided.
2. Place the slide holder(s) back into the foam insert in the specimen kit.
3. Fold Test Requisition, Pathology Report and place all paperwork into the specimen kit, between the dividing flap and the foam insert.
4. Fold down the dividing flap, place the provided cold pack on top of the flap and close the specimen kit.
5. Place the closed specimen kit into the shipping pack and apply the shipping label provided.
6. Ship via your regularly scheduled pickup or contact Customer Care to schedule a pickup.

IF YOU HAVE ANY QUESTIONS ABOUT SPECIMEN SUBMISSION, PLEASE CONTACT OUR CUSTOMER CARE TEAM AT 1.888.792.1601 OR EMAIL SPECIMENPROCUREMENT@VERACYTE.COM.

